

Record Of Service For AirSense 10 AutoSet SN 22161260341

Reference: Resindo Medika

Serviced: 03/10/2019

1 Clinical Settings

Therapy mode	AUTOSET	Product Code	37029
Serial Number	22161260341	PCB Number	(90)R370-7341(91)A8(21)63472
Software Version	SX567-0305	Bootloader Version	SX577-0200
Humidifier Software	SX556-0204	Humidifier Bootloader	SX573-0101
Motor Run Hours	8121,1 Hrs	Hours Since Last Service	8121,13 Hrs
Machine Usage Hours	8121 Hrs	Patient Hours	7943 Hrs
Product Name	AirSense 10 AutoSet	EPR Level	1 cmH2O
EPR Type	FULL TIME	Auto Pressure	4 cmH2O
Response	OFF	Set Pressure	10 cmH2O
Start Pressure	4 cmH2O	Max Pressure	19 cmH2O
Min Pressure	11 cmH2O	Start Pressure	10 cmH2O
EPR	ON	Pressure Relief	ON
Tube	SlimLine	Essentials	PLUS
Display Total Usage Hours	No	AB Filter	NO
Internal Module Fitted	NO	Ramp Time	20 Min
Pressure Unit	CMH2O	Humidifier	INTERNAL
Heated Tube	NONE	Temperature Unit	CELCIUS
Ramp Control	AUTO	Mask	NASAL
Smart Start	ON	Climate Control	AUTO
Heated Tube Enable	ON	Heated Tube Temp (C)	25
Heated Tube Temp (F)	77	MyAir Enable	NO
Humidifier Enable	ON	Humidity Level	4
Language Configuration	ENGLISH,CHINESE TRADITIONAL,CHINESE E SIMPLIFIED	Language	ENGLISH

2 Reminders

Mask	ON	Mask Reminder Date	24 Apr 2020
Mask Recur	ON	Mask Recur Period	12 Month(s)
Water Tub	OFF	Water Tub Reminder Date	10 Okt 2019
Water Tub Recur	OFF	Water Tub Recur Period	1 Month(s)
Tube	ON	Tube Reminder Date	24 Apr 2020
Tube Recur	ON	Tube Recur Period	12 Month(s)
Filter	ON	Filter Reminder Date	30 Des 2019
Filter Recur	ON	Filter Recur Period	3 Month(s)

3 Airpath Leak Test		Passed
4 Ambient Light Sensor Check		Passed
5 CAM Registration Check	Not Applicable	Passed
6 Clean Flow Generator		Completed
7 Complete Problem Report		Completed
8 Examination	External:	Passed
	Internal:	Passed
9 Flow Calibration		Passed
10 Flow Calibration Test	0 ± 1 L/min	Passed
	68 ± 3,4 L/min	Passed
	107,6 ± 5,38 L/min	Passed
	139,7 ± 6,985 L/min	Passed
11 Heated Tube Test		
12 Humidifier Test	Side Cover Connectivity:	Passed
	Hot plate:	Passed
13 Keypad Test	Key Function:	Passed
	Rotary Dial Function:	Passed
14 LCD Test		Passed
15 LED Test	Start Stop:	Passed
	Home & Rotary Dial:	Passed
16 Pressure Calibration		Passed
17 Pressure Calibration Test	0 ± 0.5 cm H2O:	Passed
	4 ± 0.5 cm H2O:	Passed
	12 ± 0.5 cm H2O:	Passed
	25 ± 0.5 cm H2O:	Passed
18 Reload Settings		Completed

19 Replace Air Filter

Completed

20 SD Card Test

Passed

21 Set Clock

22 Temperature Sensor Test

Ambient Temperature:

Passed

Absolute Humidity:

Passed

Serviced By: ResIndo Medika

Signature: _____

ResMed Service Software: 7.4.1.8498